

Step Guide on sending Glass Blood Culture Bottle for despatch via pneumatic tube transport system

Step 1: Seal the glass blood culture bottle inside the "biohazard" zip lock bag tightly



Step 2: Place inside the blood culture bottle holder



Step 3: Close the bottle holder firmly and place inside the Bubble-wrap bag/pouch



Step 4: Wrap the Bubble-wrap around the blood culture bottle in holder and seal it.



Step 5: Load into the pneumatic tube system (PTS) canister and despatch to the laboratory

