



TYPES OF SWABS IN SGH



DACRON SWAB WITHOUT MEDIA



SINGLE ESWABWITH AMIES MEDIA



DOUBLE ESWABWITH AMIES MEDIA



Molecular microbiology tests, to be sent dry.
E.g. respiratory viruses multiplex PCR.



Viral antigen tests, to be sent in sterile saline. E.g. Respiratory virus antigen IF test.



Viral isolation, to be sent in viral transport medium (VTM). E.g. Herpes simplex virus isolation.



Aerobic and anaerobic bacterial cultures

E.g. abscess, MRSA wound screening culture, VRE screening culture.



Mycology cultures

E.g. abscess, environmental sampling.



CP-CRE Screening PCR



Note: DO NOT swab chronic superficial wounds or sinus drainage as such cultures are not usef

cultures are not useful clinically - obtain a deep sample or biopsy for culture whenever possible.



MRSA Screening PCR

of nose, axilla and groin only.



Refer to instructions for MRSA Screening PCR Sample Collection.